



Comfort Audio  
HEAR THE FUTURE

**COMFORT CONTEGO**  
**Easy-to-use hearing product**  
**for a more active life**



”

## It is easy to adjust my Comfort Contego to whatever situation I am in.

Gertrud and I travel a lot, and it is such a joy now to be able to sit and chat during our travels. In the car the humming and engine noise is filtered out, and I can hear clearly what Gertrud is saying. It also feels very natural and it is great not having to ask her to repeat herself the whole time.

The Comfort Contego is simple to use and can easily be adjusted to whatever situation I am in, just by switching it from directional to omnidirectional mode. For instance, when we are eating dinner with friends and family, I simply set the microphone mode to omnidirectional and place the transmitter on the table to pick up speech. As I can hear what everyone is saying, I can follow and take part in conversations and discussions in a completely different way than before. When Gertrud and I are alone, she wears the transmitter clipped to her top, so when we go shopping or for a walk in the park we can easily talk while we are on the move. I wear the Comfort Contego around my neck in all situations, and this is why I appreciate its flexible and neat design.

Kjell, pensioner in Malmö, Sweden

”









”

## The new hearing product has made our lives easier.

We can now hold a conversation without any problem; it feels easy and natural. And it is wonderful not having to repeat oneself too many times. Communicating with family and friends has become so much easier. It used to be hard for Kjell to participate fully as there were many comments he did not catch. Now he can hear what everyone is saying, even when we are many people together. It is of course a big step forward for us both that he can enjoy other people's company just like the rest of us.

Kjell always takes his Comfort Contego with him when he leaves the house. I feel less anxious knowing that now he can hear the traffic and can tell for certain if a car is beeping its horn. We enjoy an active and outside life style, being out in the forest and the countryside, and our experience of nature has been enhanced. It is amazing that Kjell can now even hear the birds singing so we can enjoy it together.

When the Comfort Contego came into our lives it was in many ways a moment of realisation for me too, and it is fantastic how it has made it possible for Kjell to take a more active part in life.

Gertrud, partner to Kjell

Here, Kjell and Gertrud describe what a great help their equipment is in their everyday lives, as it is for many other satisfied Comfort Contego users. The people depicted are not those quoted.

”

The Comfort Contego uses a unique noise free digital radio transmission. It helps you catch the details of a conversation regardless of whether it is between many people or just a few.



## Seize the moment

There are times when you need extra help in addition to your hearing aid.

This is why we at Comfort Audio have developed the Comfort Contego, a digital wireless hearing product that helps people like you, with a hearing loss, to fully participate in all situations with family and friends. The Comfort Contego is flexible and can be used with a neck loop if you wear a hearing aid or can be used without hearing aids by using headphones. It is very simple to use, with its easy-to-operate volume controls and clear display. It is based on advanced technology giving you a whole new way to benefit from speech.

Comfort Audio was first in the world to develop digital hearing products. Early on we started researching into new ways of helping people with impaired hearing. Our dedicated work manifests itself in products with features that are unique in the industry, making us the world leader in technology.







The transmitter on the table picks up speech and send it to the receiver. Hearing what everyone is saying brings a feeling of joy and a sense of participation.





## Try it out and experience the difference

The Comfort Contego consists of a transmitter and a receiver, both units are uniquely equipped with microphones. Unique digital radio transmission gives excellent sound quality with natural speech reproduction. Distracting noise from the background and transmission is reduced as a result of shorter listening distances, making speech more distinct.

We could talk for ages on the technology behind the Comfort Contego and what it can do for you. But only when you test it for yourself can you really understand how the Comfort Contego makes a difference and helps you to participate in all imaginable situations.



If you have a hearing aid with a T-mode, you can use the Comfort Contego together with a neck loop. Otherwise you can use it with headphones or earplugs.


## Comfort Contego in different situations

You have your hearing product with you in many different situations and use it to participate in all conversations.


### Comfort Contego in brief:

- Easy to use
- Excellent sound quality without time lag
- Clear display
- Eavesdrop secure
- Long operating time (up to 20 hours)


▶ Sound transmission    ))) Speech

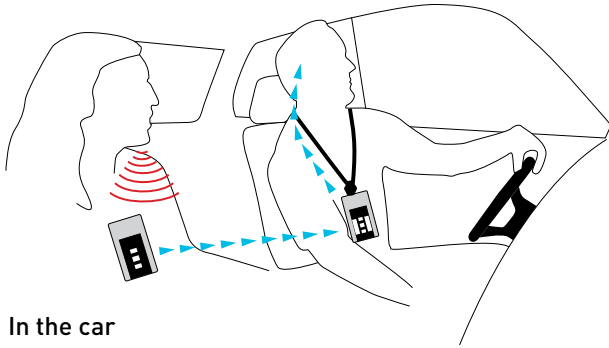


In the car

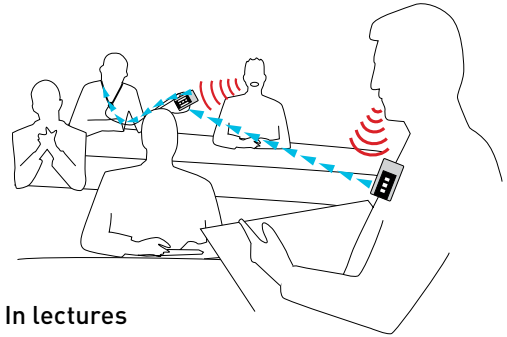


In lectures

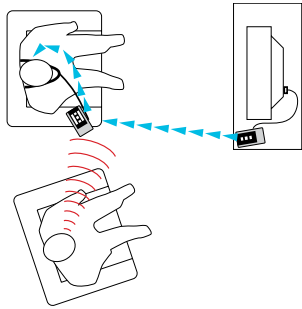


Watching TV

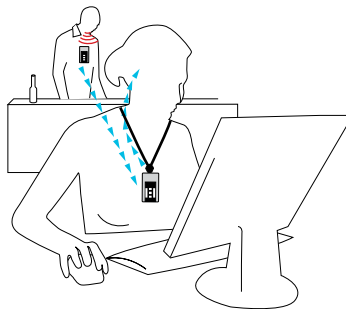


At home

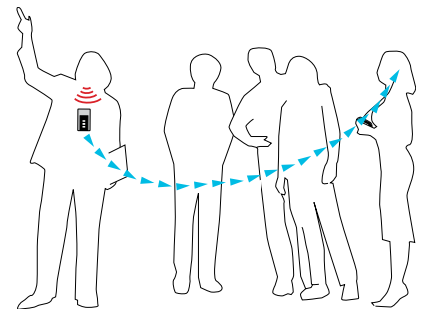


On holiday

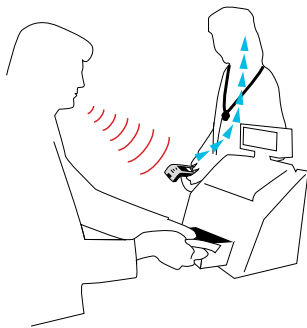

Shopping

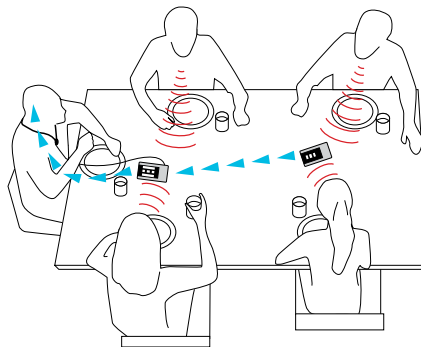

At the dinner table

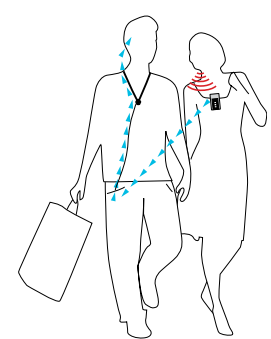

Walking

# Comfort Contego



## Technical data

Range: Up to 30 m

Dimensions: 22 x 45 x 98 mm

Weight: 64 g / 62 g respectively

Operating time: Up to 20 hours

Battery: Rechargeable Li-Ion

Antenna: Built-in

Since 1994 Comfort Audio has been developing and manufacturing hearing products for people with a hearing loss. We shall through leading technology and innovative solutions, make people with a hearing loss feel comfortable to interact in all situations through all stages of life.

We are dedicated, curious, highly technically motivated and are driven by passion and joy. This has made us world leaders in hearing product technology. Our products offer great flexibility and have been developed for working life, school, public places and private life. The needs of the individual are always in focus and we have a genuine desire to improve the quality of life for people with a hearing loss.